ÁCIDO ÚRICO ENZIMÁTICO

CÓDIGO	VOLUME (mL)	Nº. TESTES
100/100-100	100	500
100/100-200	200	1000

ITEM NAME	#	AUR
-----------	---	-----

DATA INFORMATIONS

UNITS mg/dL
DECIMALS 2

ANALYSIS

TYPE END
MAIN W.LENGTH 1 505
SUB W LENGTH 2

METHOD ENZIMÁTICO TRINDER

CORRELAÇÃO

Y =	SLOPE	X +	INTER
	1		0

CALIBRATION

TYPE STANDARD LINEAR

#1	@	#4	
#2		#5	
#3		#6	

NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM	2,5	7,0	1.5	6.0
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM	1,5	20,0
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

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NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM	3.5	4.8	3.5	4.8
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM	1,0	6,0
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

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UNITS U/L
DECIMALS 0

ANALYSIS

TYPE RATE
MAIN W.LENGTH 1 405
SUB W LENGTH 2
METHOD CNP

CORRELAÇÃO

Y =	SLOPE	X +	INTER
	1		0

CALIBRATION

TYPE STANDARD LINEAR

#1	@	#4	
#2		#5	
#3		#6	

NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM	25	125	25	125
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM	5	8000
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

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NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
	#	#	#	#
SERUM				
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM	20	800
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

ASPIRATION

KIND SINGLE DOUBLE

	VOLUME
SAMPLE	2
REAGENT 1 VOL µL	160
REAGENT 2	40

THIRD MIX OFF ON

R1 BLANK WATER BLANK R1-BLANK1

MONITOR

0 LEVEL POINT	1
SPAN	3

DATA PROCESS

READ

	START	END
MAIN	34	42
SUB		

ABSORBANCE LIMIT

LOW	HIGH
-0.010	3.000

FACTOR

BLANK CORRECTION	1
------------------	---

ENDPOINT LIMIT	
LINEAR CHECK (%)	20

PROZONE CHECK

	START	END	LIMIT (%)
FIRST			
SECOND			
THIRD			

LOW HIGH
 LOW HIGH

AUTO RERUN CONDITION (ABSORBANCE)

ABSORBANCE RANGE LOWER ON OFF
HIGHER ON OFF

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NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM	0	0,25	0	0,25
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM	0	15
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

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NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM	0	1.1	0	1.1
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM	0	15
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

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NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM	8.5	10	8.5	10
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM	0	20
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

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NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM	98	110	98	110
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM		
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

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NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM	0	25	0	25
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM	1	500
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

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NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM	24	195	0	170
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM	5	2000
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

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NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM	0	240	0	240
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM	10	800
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

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NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM	35	150	35	150
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM	5	150
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

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NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM	59	158	37	145
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM		
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

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UNITS U/I
DECIMALS 0

ANALYSIS

TYPE RATE
MAIN W. LENGTH 1 405
SUB W LENGTH 2
METHOD DGKC

CORRELAÇÃO

Y =	SLOPE	X +	INTER
	1		0

CALIBRATION

TYPE LINEAR

STANDARD			
#1	@	#4	
#2		#5	
#3		#6	

NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM	0	240	0	270
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM	5	700
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

ASPIRATION

KIND SINGLE DOUBLE

VOLUME	
SAMPLE	4
REAGENT 1 VOL µL	160
REAGENT 2	40

THIRD MIX OFF ON

R1 BLANK WATER BLANK R1-BLANK1

MONITOR

0 LEVEL POINT	1
SPAN	3

DATA PROCESS

READ

	START	END
MAIN	38	50
SUB		

ABSORBANCE LIMIT

LOW	HIGH
-0.010	0.900

FACTOR

BLANK CORRECTION	1
------------------	---

ENDPOINT LIMIT	
LINEAR CHECK (%)	20

PROZONE CHECK

	START	END	LIMIT (%)
FIRST			
SECOND			
THIRD			

LOW HIGH

LOW HIGH

AUTO RERUN CONDITION (ABSORBANCE)

ABSORBANCE RANGE LOWER ON OFF
HIGHER ON OFF

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NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM	2.5	5.0	2.5	5.0
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM		
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

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NORMAL RANGE

	MALE		FEMALE	
	LOW #	HIGH #	LOW #	HIGH #
SERUM				
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM	6,0	160,0
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

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NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM	205	285	205	285
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM	2	800
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

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NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM	11	50	7	32
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM	1	250
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

ASPIRATION

KIND SINGLE DOUBLE

	VOLUME
SAMPLE	20
REAGENT 1 VOL µL	160
REAGENT 2	40

THIRD MIX OFF ON

R1 BLANK WATER BLANK R1-BLANK1

MONITOR

0 LEVEL POINT	1
SPAN	3

DATA PROCESS

READ

	START	END
MAIN	38	50
SUB		

ABSORBANCE LIMIT

LOW	HIGH
-0.010	3.000

FACTOR

BLANK CORRECTION

ENDPOINT LIMIT	
LINEAR CHECK (%)	20

PROZONE CHECK

	START	END	LIMIT (%)
FIRST			
SECOND			
THIRD			

LOW HIGH

LOW HIGH

AUTO RERUN CONDITION (ABSORBANCE)

ABSORBANCE RANGE LOWER ON OFF
HIGHER ON OFF

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NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM	60	99	60	99
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM	35	500
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

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NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM	0	32	0	42
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM	10	440
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

ASPIRATION

KIND SINGLE DOUBLE

	VOLUME
SAMPLE	20
REAGENT 1 VOL µL	160
REAGENT 2	40

THIRD MIX OFF ON

R1 BLANK WATER BLANK R1-BLANK1

MONITOR

0 LEVEL POINT	1
SPAN	3

DATA PROCESS

READ

	START	END
MAIN	38	50
SUB		

ABSORBANCE LIMIT

LOW	HIGH
-0.010	0.900

FACTOR

BLANK CORRECTION	1
------------------	---

ENDPOINT LIMIT	
LINEAR CHECK (%)	40

PROZONE CHECK

	START	END	LIMIT (%)
FIRST			
SECOND			
THIRD			

LOW HIGH

LOW HIGH

AUTO RERUN CONDITION (ABSORBANCE)

ABSORBANCE RANGE LOWER ON OFF
HIGHER ON OFF

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NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM	0	32	0	42
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM	10	350
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

ASPIRATION

KIND SINGLE DOUBLE

	VOLUME
SAMPLE	20
REAGENT 1 VOL µL	160
REAGENT 2	40

THIRD MIX OFF ON

R1 BLANK WATER BLANK R1-BLANK1

MONITOR

0 LEVEL POINT	1
SPAN	3

DATA PROCESS

READ

	START	END
MAIN	38	50
SUB		

ABSORBANCE LIMIT

LOW	HIGH
-0.010	0.900

FACTOR

BLANK CORRECTION	1
------------------	---

ENDPOINT LIMIT	
LINEAR CHECK (%)	40

PROZONE CHECK

	START	END	LIMIT (%)
FIRST			
SECOND			
THIRD			

LOW HIGH
 LOW HIGH

AUTO RERUN CONDITION (ABSORBANCE)

ABSORBANCE RANGE LOWER ON OFF
HIGHER ON OFF

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NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM	4.5	19.8	4.5	19.8
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM		
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

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NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM	225	450	225	450
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM	40	2000
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

ASPIRATION

KIND SINGLE DOUBLE

	VOLUME
SAMPLE	4
REAGENT 1 VOL µL	160
REAGENT 2	40

THIRD MIX OFF ON

R1 BLANK WATER BLANK R1-BLANK1

MONITOR

0 LEVEL POINT	1
SPAN	3

DATA PROCESS

READ

	START	END
MAIN	38	50
SUB		

ABSORBANCE LIMIT

LOW	HIGH
-0.010	3.000

FACTOR

BLANK CORRECTION	1
------------------	---

ENDPOINT LIMIT	
LINEAR CHECK (%)	20

PROZONE CHECK

	START	END	LIMIT (%)
FIRST			
SECOND			
THIRD			

LOW HIGH

LOW HIGH

AUTO RERUN CONDITION (ABSORBANCE)

ABSORBANCE RANGE LOWER ON OFF
HIGHER ON OFF

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NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
	#	#	#	#
SERUM				
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM	20	300
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

ASPIRATION

KIND SINGLE DOUBLE

	VOLUME
SAMPLE	2
REAGENT 1 VOL µL	160
REAGENT 2	40

THIRD MIX OFF ON

R1 BLANK WATER BLANK R1-BLANK1

MONITOR

0 LEVEL POINT	1
SPAN	3

DATA PROCESS

READ

	START	END
MAIN	40	42
SUB		

ABSORBANCE LIMIT

LOW	HIGH
-0.010	3.000

FACTOR

BLANK CORRECTION	1
------------------	---

ENDPOINT LIMIT	
LINEAR CHECK (%)	20

PROZONE CHECK

	START	END	LIMIT (%)
FIRST			
SECOND			
THIRD			

LOW HIGH
 LOW HIGH

AUTO RERUN CONDITION (ABSORBANCE)

ABSORBANCE RANGE LOWER ON OFF
HIGHER ON OFF

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NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM	1.9	2.5	1.9	2.5
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
	SERUM	0,5
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

ASPIRATION

KIND SINGLE DOUBLE

VOLUME	
SAMPLE	2
REAGENT 1 VOL µL	200

THIRD MIX OFF ON

R1 BLANK WATER BLANK R1-BLANK1

MONITOR

0 LEVEL POINT	1
SPAN	3

DATA PROCESS

READ

	START	END
	MAIN	25
SUB		

ABSORBANCE LIMIT

LOW	HIGH
-0.010	3.000

FACTOR

BLANK CORRECTION	1
------------------	---

ENDPOINT LIMIT	0.020
LINEAR CHECK (%)	

PROZONE CHECK

	START	END	LIMIT (%)
FIRST			
SECOND			
THIRD			

LOW HIGH

LOW HIGH

AUTO RERUN CONDITION (ABSORBANCE)

ABSORBANCE RANGE LOWER ON OFF
HIGHER ON OFF

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NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM	#	#	#	#
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM	1,0	150,0
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

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NORMAL RANGE

	MALE		FEMALE	
	LOW #	HIGH #	LOW #	HIGH #
SERUM				
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM	0,05	10,0
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

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NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM	6.5	8.0	6.5	8.0
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
	SERUM	0,5
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

ASPIRATION

KIND SINGLE DOUBLE

	VOLUME
SAMPLE	2
REAGENT 1 VOL µL	200

THIRD MIX OFF ON

R1 BLANK WATER BLANK R1-BLANK1

MONITOR

0 LEVEL POINT	1
SPAN	3

DATA PROCESS

READ

	START	END
MAIN	33	34
SUB		

ABSORBANCE LIMIT

LOW	HIGH
-0.010	3.000

FACTOR

BLANK CORRECTION	1
------------------	---

ENDPOINT LIMIT	0.020
LINEAR CHECK (%)	

PROZONE CHECK

	START	END	LIMIT (%)
FIRST			
SECOND			
THIRD			

LOW HIGH

LOW HIGH

AUTO RERUN CONDITION (ABSORBANCE)

ABSORBANCE RANGE LOWER ON OFF
HIGHER ON OFF

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NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM				
URINE	150	450	150	450
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
	SERUM	
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

ASPIRATION

KIND SINGLE DOUBLE

	VOLUME
SAMPLE	4
REAGENT 1 VOL µL	200

THIRD MIX OFF ON

R1 BLANK WATER BLANK R1-BLANK1

MONITOR

0 LEVEL POINT	1
SPAN	3

DATA PROCESS

READ

	START	END
MAIN	33	34
SUB		

ABSORBANCE LIMIT

LOW	HIGH
-0.010	3.000

FACTOR

BLANK CORRECTION	1
------------------	---

ENDPOINT LIMIT	0.020
LINEAR CHECK (%)	

PROZONE CHECK

	START	END	LIMIT (%)
FIRST			
SECOND			
THIRD			

LOW HIGH

LOW HIGH

AUTO RERUN CONDITION (ABSORBANCE)

ABSORBANCE RANGE LOWER ON OFF
HIGHER ON OFF

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NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM	150	200	150	200
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM	10	1100
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

ASPIRATION

KIND SINGLE DOUBLE

	VOLUME
SAMPLE	2
REAGENT 1 VOL µL	200

THIRD MIX OFF ON

R1 BLANK WATER BLANK R1-BLANK1

MONITOR

0 LEVEL POINT	1
SPAN	3

DATA PROCESS

READ

	START	END
MAIN	34	35
SUB		

ABSORBANCE LIMIT

LOW	HIGH
-0.010	1.500

FACTOR

BLANK CORRECTION	1
------------------	---

ENDPOINT LIMIT	0.010
LINEAR CHECK (%)	

PROZONE CHECK

	START	END	LIMIT (%)
FIRST			
SECOND			
THIRD			

LOW HIGH

LOW HIGH

AUTO RERUN CONDITION (ABSORBANCE)

ABSORBANCE RANGE LOWER ON OFF
HIGHER ON OFF

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NORMAL RANGE

	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
SERUM	15	45	15	45
URINE				
PLASMA				
CSF				
DIALYSIS				
OTHER				

AUTO RERUN SW

ON OFF

AUTO RERUN RANGE (RESULTS)

ON OFF ON OFF

	LOWER	HIGHER
SERUM	10	300
URINE		
PLASMA		
CSF		
DIALYSIS		
OTHER		

ASPIRATION

KIND SINGLE DOUBLE

	VOLUME
SAMPLE	2
REAGENT 1 VOL µL	160
REAGENT 2	40

THIRD MIX OFF ON

R1 BLANK WATER BLANK R1-BLANK1

MONITOR

0 LEVEL POINT	1
SPAN	3

DATA PROCESS

READ

	START	END
MAIN	36	44
SUB		

ABSORBANCE LIMIT

LOW	HIGH
-0.010	3.000

FACTOR

BLANK CORRECTION	1
------------------	---

ENDPOINT LIMIT	
LINEAR CHECK (%)	20

PROZONE CHECK

	START	END	LIMIT (%)
FIRST			
SECOND			
THIRD			

LOW HIGH
 LOW HIGH

AUTO RERUN CONDITION (ABSORBANCE)

ABSORBANCE RANGE LOWER ON OFF
HIGHER ON OFF

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