ÁCIDO ÚRICO ENZIMÁTICO

CÓDIGO	VOLUME (mL)	Nº. TESTES
100/100-100	100	400
100/100-200	200	800

TEST NAME	ÁCIDO ÚRICO
TEST	AU
TEST BAR CODE	#
TEST TYPE	Ponto Final
CURVE TYPE	Lineal Suprimida
UNITS	mg/dL
NO. OF DECIMAL PLACES	1
PRIMARY WAVELENGTH	510 nm
SECONDARY WAVELENGTH	-
READ TIME/INTERVAL	20
SAMPLE BLANK	NÃO
FACTOR	**
CALIBRATION INTERVAL	*
NORMALIZATION INTERVAL	-
NO. OF CALIBRATORS	2
NO. OF REPLICATES	2
LOW BLANK A LIMIT	-0.100
HIGH BLANK A LIMIT	0.500
LOW A LIMIT	0.000
HIGH A LIMIT	2.400
LOW NORMAL	#
HIGH NORMAL	#
LINEARITY LIMIT	20.0 mg/dL
CURVES S.D. LIMIT	0.500

TEST NAME	ÁCIDO ÚRICO
TEST	AU
TEST BAR CODE	#
SAMPLE VOLUME	5 µL
SAMPLE DILUENT	-
PREDILUTION RATIO	-
RERUN DILUTION RATIO	2
PREDILUTION	1
REAGENT DILUENT	-
REAGENT 1 VOLUME	250 µL
DILUENT VOLUME	-
BAR CODE	AUR
LAG TIME	300 seg.
REAGENT 2 VOLUME	-
DILUENT VOLUME	-
BAR CODE	-
LAG TIME	-

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TEST NAME	ALBUMINA
TEST	ALB
TEST BAR CODE	#
TEST TYPE	Ponto Final
CURVE TYPE	Lineal Suprimida
UNITS	g/dL
NO. OF DECIMAL PLACES	1
PRIMARY WAVELENGTH	600 nm
SECONDARY WAVELENGTH	-
READ TIME/INTERVAL	20
SAMPLE BLANK	NÃO
FACTOR	**
CALIBRATION INTERVAL	*
NORMALIZATION INTERVAL	-
NO. OF CALIBRATORS	2
NO. OF REPLICATES	2
LOW BLANK A LIMIT	-0.010
HIGH BLANK A LIMIT	0.400
LOW A LIMIT	0.000
HIGH A LIMIT	2.400
LOW NORMAL	#
HIGH NORMAL	#
LINEARITY LIMIT	6.0 g/dL
CURVES S.D. LIMIT	0.500

TEST NAME	ALBUMINA
TEST	ALB
TEST BAR CODE	#
SAMPLE VOLUME	3 µL
SAMPLE DILUENT	-
PREDILUTION RATIO	1
RERUN DILUTION RATIO	1
PREDILUTION	-
REAGENT DILUENT	-
REAGENT 1 VOLUME	400 µL
DILUENT VOLUME	-
BAR CODE	ALBU
LAG TIME	120 seg.
REAGENT 2 VOLUME	-
DILUENT VOLUME	-
BAR CODE	-
LAG TIME	-

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TEST NAME	BILD
TEST	BID
TEST BAR CODE	#
TEST TYPE	Ponto Final
CURVE TYPE	Lineal
	Suprimida
UNITS	mg/dL
NO. OF DECIMAL PLACES	2
PRIMARY WAVELENGTH	540 nm
SECONDARY WAVELENGTH	-
READ TIME/INTERVAL	20
SAMPLE BLANK	Sim
FACTOR	@
CALIBRATION INTERVAL	#
NORMALIZATION INTERVAL	-
NO. OF CALIBRATORS	2
NO. OF REPLICATES	2
LOW BLANK A LIMIT	-0.100
HIGH BLANK A LIMIT	0.500
LOW A LIMIT	-0.100
HIGH A LIMIT	2.400
LOW NORMAL	#
HIGH NORMAL	#
LINEARITY LIMIT	15.0 mg/dL
CURVES S.D. LIMIT	-

TEST NAME	BILD
TEST	BID
TEST BAR CODE	#
SAMPLE VOLUME	10 µL
SAMPLE DILUENT	-
PREDILUTION RATIO	1
RERUN DILUTION RATIO	2
PREDILUTION	-
REAGENT DILUENT	-
REAGENT 1 VOLUME	200 µL
DILUENT VOLUME	-
BAR CODE	BD1A
LAG TIME	10 seg.
REAGENT 2 VOLUME	6 µL
DILUENT VOLUME	-
BAR CODE	BD2A
LAG TIME	300 seg

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TEST NAME	BILT
TEST	BIT
TEST BAR CODE	#
TEST TYPE	Ponto Final
CURVE TYPE	Lineal Suprimida
UNITS	mg/dL
NO. OF DECIMAL PLACES	2
PRIMARY WAVELENGTH	540 nm
SECONDARY WAVELENGTH	-
READ TIME/INTERVAL	20
SAMPLE BLANK	Sim
FACTOR	@
CALIBRATION INTERVAL	#
NORMALIZATION INTERVAL	-
NO. OF CALIBRATORS	2
NO. OF REPLICATES	2
LOW BLANK A LIMIT	-0.100
HIGH BLANK A LIMIT	0.500
LOW A LIMIT	-0.100
HIGH A LIMIT	2.400
LOW NORMAL	#
HIGH NORMAL	#
LINEARITY LIMIT	15.0 mg/dL
CURVES S.D. LIMIT	0.500

TEST NAME	BILT
TEST	BIT
TEST BAR CODE	#
SAMPLE VOLUME	10 µL
SAMPLE DILUENT	-
PREDILUTION RATIO	1
RERUN DILUTION RATIO	2
PREDILUTION	-
REAGENT DILUENT	-
REAGENT 1 VOLUME	200 µL
DILUENT VOLUME	-
BAR CODE	BT1A
LAG TIME	10 seg.
REAGENT 2 VOLUME	6 µL
DILUENT VOLUME	-
BAR CODE	BT2A
LAG TIME	300 seg

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TEST NAME	CÁLCIO ARSE
TEST	CAA
TEST BAR CODE	#
TEST TYPE	Ponto Final
CURVE TYPE	Lineal
	Suprimida
UNITS	mg/dL
NO. OF DECIMAL PLACES	1
PRIMARY WAVELENGTH	600 nm
SECONDARY WAVELENGTH	-
READ TIME/INTERVAL	10
SAMPLE BLANK	Não
FACTOR	**
CALIBRATION INTERVAL	*
NORMALIZATION INTERVAL	-
NO. OF CALIBRATORS	2
NO. OF REPLICATES	2
LOW BLANK A LIMIT	-0.100
HIGH BLANK A LIMIT	2.500
LOW A LIMIT	-0.100
HIGH A LIMIT	2.400
LOW NORMAL	#
HIGH NORMAL	#
LINEARITY LIMIT	20.0 mg/dL
CURVES S.D. LIMIT	0.500

TEST NAME	CÁLCIO ARSE
TEST	CAA
TEST BAR CODE	#
SAMPLE VOLUME	3 µL
SAMPLE DILUENT	-
PREDILUTION RATIO	1
RERUN DILUTION RATIO	1
PREDILUTION	-
REAGENT DILUENT	-
REAGENT 1 VOLUME	300 µL
DILUENT VOLUME	-
BAR CODE	CA1A
LAG TIME	180 seg.
REAGENT 2 VOLUME	-
DILUENT VOLUME	-
BAR CODE	-
LAG TIME	-

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TEST NAME	CÁLCIO CRES
TEST	CACR
TEST BAR CODE	#
TEST TYPE	Ponto Final
CURVE TYPE	Lineal
	Suprimida
UNITS	mg/dL
NO. OF DECIMAL PLACES	1
PRIMARY WAVELENGTH	600 nm
SECONDARY WAVELENGTH	-
READ TIME/INTERVAL	10
SAMPLE BLANK	Não
FACTOR	*
CALIBRATION INTERVAL	**
NORMALIZATION INTERVAL	-
NO. OF CALIBRATORS	2
NO. OF REPLICATES	2
LOW BLANK A LIMIT	-0.10
HIGH BLANK A LIMIT	2.50
LOW A LIMIT	0.00
HIGH A LIMIT	2.00
LOW NORMAL	#
HIGH NORMAL	#
LINEARITY LIMIT	20.0 mg/dL
CURVES S.D. LIMIT	*

TEST NAME	CÁLCIO CRE
TEST	CACR
TEST BAR CODE	#
SAMPLE VOLUME	3 µL
SAMPLE DILUENT	-
PREDILUTION RATIO	1
RERUN DILUTION RATIO	1
PREDILUTION	-
REAGENT DILUENT	-
REAGENT 1 VOLUME	300 µL
DILUENT VOLUME	-
BAR CODE	CAASX
LAG TIME	180 seg.
REAGENT 2 VOLUME	-
DILUENT VOLUME	-
BAR CODE	-
LAG TIME	-

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TEST NAME	CLORETOS
TEST	CLO
TEST BAR CODE	#
TEST TYPE	Ponto Final
CURVE TYPE	Lineal
	Suprimida
UNITS	mEq/L
NO. OF DECIMAL PLACES	1
PRIMARY WAVELENGTH	510 nm
SECONDARY WAVELENGTH	-
READ TIME/INTERVAL	10
SAMPLE BLANK	Não
FACTOR	**
CALIBRATION INTERVAL	*
NORMALIZATION INTERVAL	-
NO. OF CALIBRATORS	2
NO. OF REPLICATES	2
LOW BLANK A LIMIT	-0.100
HIGH BLANK A LIMIT	0.600
LOW A LIMIT	-0.100
HIGH A LIMIT	2.400
LOW NORMAL	#
HIGH NORMAL	#
LINEARITY LIMIT	150.0 mEq/L
CURVES S.D. LIMIT	0.500

TEST NAME	CLORETOS
TEST	CLO
TEST BAR CODE	#
SAMPLE VOLUME	3 µL
SAMPLE DILUENT	-
PREDILUTION RATIO	1
RERUN DILUTION RATIO	1
PREDILUTION	-
REAGENT DILUENT	-
REAGENT 1 VOLUME	400 µL
DILUENT VOLUME	-
BAR CODE	CL1A
LAG TIME	290 seg.
REAGENT 2 VOLUME	-
DILUENT VOLUME	-
BAR CODE	-
LAG TIME	-

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TEST NAME	COLESTEROL
TEST	COL
TEST BAR CODE	#
TEST TYPE	Ponto Final
CURVE TYPE	Lineal
	Suprimida
UNITS	mg/dL
NO. OF DECIMAL PLACES	1
PRIMARY WAVELENGTH	510 nm
SECONDARY WAVELENGTH	-
READ TIME/INTERVAL	20
SAMPLE BLANK	-
FACTOR	*
CALIBRATION INTERVAL	**
NORMALIZATION INTERVAL	-
NO. OF CALIBRATORS	2
NO. OF REPLICATES	2
LOW BLANK A LIMIT	-0.100
HIGH BLANK A LIMIT	0.500
LOW A LIMIT	-0.100
HIGH A LIMIT	2.400
LOW NORMAL	#
HIGH NORMAL	#
LINEARITY LIMIT	800.0 mg/dL
CURVES S.D. LIMIT	10

TEST NAME	COLESTEROL
TEST	COL
TEST BAR CODE	#
SAMPLE VOLUME	3 µL
SAMPLE DILUENT	-
PREDILUTION RATIO	1
RERUN DILUTION RATIO	2
PREDILUTION	-
REAGENT DILUENT	-
REAGENT 1 VOLUME	300 µL
DILUENT VOLUME	-
BAR CODE	COLB
LAG TIME	300 seg.
REAGENT 2 VOLUME	-
DILUENT VOLUME	-
BAR CODE	-
LAG TIME	-

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TEST NAME	HDL DIRETO
TEST	HDL
TEST BAR CODE	#
TEST TYPE	Ponto Final
CURVE TYPE	Lineal
	Suprimida
UNITS	mg/dL
NO. OF DECIMAL PLACES	1
PRIMARY WAVELENGTH	570 nm
SECONDARY WAVELENGTH	-
READ TIME/INTERVAL	10
SAMPLE BLANK	-
FACTOR	**
CALIBRATION INTERVAL	*
NORMALIZATION INTERVAL	-
NO. OF CALIBRATORS	2
NO. OF REPLICATES	2
LOW BLANK A LIMIT	-0.100
HIGH BLANK A LIMIT	0.500
LOW A LIMIT	-0.100
HIGH A LIMIT	2.400
LOW NORMAL	#
HIGH NORMAL	#
LINEARITY LIMIT	150.0 mg/dL
CURVES S.D. LIMIT	10

TEST NAME	HDL DIRETO
TEST	HDL
TEST BAR CODE	#
SAMPLE VOLUME	4 µL
SAMPLE DILUENT	-
PREDILUTION RATIO	1
RERUN DILUTION RATIO	2
PREDILUTION	-
REAGENT DILUENT	-
REAGENT 1 VOLUME	300 µL
DILUENT VOLUME	-
BAR CODE	HDL1
LAG TIME	290 seg.
REAGENT 2 VOLUME	100 µL
DILUENT VOLUME	-
BAR CODE	HDL2
LAG TIME	290 seg.

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TEST NAME	HDL PREC
TEST	HDLP
TEST BAR CODE	#
TEST TYPE	Ponto Final
CURVE TYPE	Lineal
	Suprimida
UNITS	mg/dL
NO. OF DECIMAL PLACES	1
PRIMARY WAVELENGTH	510 nm
SECONDARY WAVELENGTH	-
READ TIME/INTERVAL	20
SAMPLE BLANK	-
FACTOR	**
CALIBRATION INTERVAL	*
NORMALIZATION INTERVAL	-
NO. OF CALIBRATORS	2
NO. OF REPLICATES	2
LOW BLANK A LIMIT	-0.100
HIGH BLANK A LIMIT	0.500
LOW A LIMIT	-0.100
HIGH A LIMIT	2.400
LOW NORMAL	#
HIGH NORMAL	#
LINEARITY LIMIT	150.0 mg/dL
CURVES S.D. LIMIT	10

TEST NAME	HDL PREC
TEST	HDLP
TEST BAR CODE	#
SAMPLE VOLUME	25 µL
SAMPLE DILUENT	-
PREDILUTION RATIO	1
RERUN DILUTION RATIO	2
PREDILUTION	-
REAGENT DILUENT	-
REAGENT 1 VOLUME	250 µL
DILUENT VOLUME	-
BAR CODE	HDLB
LAG TIME	300 seg.
REAGENT 2 VOLUME	-
DILUENT VOLUME	-
BAR CODE	-
LAG TIME	-

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TEST NAME	CREATININA
TEST	CREA
TEST BAR CODE	#
TEST TYPE	Cinético
CURVE TYPE	Lineal
	Suprimida
UNITS	mg/dL
NO. OF DECIMAL PLACES	1
PRIMARY WAVELENGTH	510 nm
SECONDARY WAVELENGTH	-
READ TIME/INTERVAL	90
SAMPLE BLANK	NÃO
FACTOR	**
CALIBRATION INTERVAL	*
NORMALIZATION INTERVAL	-
NO. OF CALIBRATORS	2
NO. OF REPLICATES	2
LOW BLANK A LIMIT	-0.100
HIGH BLANK A LIMIT	2.500
LOW A LIMIT	-0.100
HIGH A LIMIT	2.400
LOW NORMAL	#
HIGH NORMAL	#
LINEARITY LIMIT	0.0 mg/dL
CURVES S.D. LIMIT	0.300

TEST NAME	CREATININA
TEST	CREA
TEST BAR CODE	#
SAMPLE VOLUME	30 µL
SAMPLE DILUENT	-
PREDILUTION RATIO	1
RERUN DILUTION RATIO	2
PREDILUTION	-
REAGENT DILUENT	-
REAGENT 1 VOLUME	300 µL
DILUENT VOLUME	-
BAR CODE	CR1A
LAG TIME	60 seg.
REAGENT 2 VOLUME	-
DILUENT VOLUME	-
BAR CODE	-
LAG TIME	-

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TEST NAME	CFREATININA 2R
TEST	CREA2R
TEST BAR CODE	#
TEST TYPE	Cinético
CURVE TYPE	Lineal
	Suprimida
UNITS	mg/dL
NO. OF DECIMAL PLACES	1
PRIMARY WAVELENGTH	510 nm
SECONDARY WAVELENGTH	-
READ TIME/INTERVAL	90
SAMPLE BLANK	-
FACTOR	**
CALIBRATION INTERVAL	*
NORMALIZATION INTERVAL	-
NO. OF CALIBRATORS	2
NO. OF REPLICATES	2
LOW BLANK A LIMIT	-0.100
HIGH BLANK A LIMIT	0.500
LOW A LIMIT	-0.100
HIGH A LIMIT	2.400
LOW NORMAL	#
HIGH NORMAL	#
LINEARITY LIMIT	10.0 mg/dL
CURVES S.D. LIMIT	0.300

TEST NAME	CEATININA 2R
TEST	CREA2R
TEST BAR CODE	#
SAMPLE VOLUME	30 µL
SAMPLE DILUENT	-
PREDILUTION RATIO	1
RERUN DILUTION RATIO	2
PREDILUTION	-
REAGENT DILUENT	-
REAGENT 1 VOLUME	240 µL
DILUENT VOLUME	-
BAR CODE	HDL1
LAG TIME	10 seg.
REAGENT 2 VOLUME	60 µL
DILUENT VOLUME	-
BAR CODE	HDL2
LAG TIME	60 seg.

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TEST NAME	FERRO FER
TEST	FEFER
TEST BAR CODE	#
TEST TYPE	Ponto Final
CURVE TYPE	Lineal
	Suprimida
UNITS	µg/dL
NO. OF DECIMAL PLACES	1
PRIMARY WAVELENGTH	565 nm
SECONDARY WAVELENGTH	-
READ TIME/INTERVAL	20
SAMPLE BLANK	-
FACTOR	**
CALIBRATION INTERVAL	*
NORMALIZATION INTERVAL	-
NO. OF CALIBRATORS	2
NO. OF REPLICATES	2
LOW BLANK A LIMIT	-0.100
HIGH BLANK A LIMIT	0.500
LOW A LIMIT	-0.100
HIGH A LIMIT	2.400
LOW NORMAL	#
HIGH NORMAL	#
LINEARITY LIMIT	500.0 µg/dL
CURVES S.D. LIMIT	20

TEST NAME	FERRO FER
TEST	FEFER
TEST BAR CODE	#
SAMPLE VOLUME	30 µL
SAMPLE DILUENT	-
PREDILUTION RATIO	-
RERUN DILUTION RATIO	1
PREDILUTION	-
REAGENT DILUENT	-
REAGENT 1 VOLUME	220 µL
DILUENT VOLUME	-
BAR CODE	FEF
LAG TIME	300 seg.
REAGENT 2 VOLUME	-
DILUENT VOLUME	-
BAR CODE	-
LAG TIME	-

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TEST NAME	FE BI
TEST	FEBI
TEST BAR CODE	#
TEST TYPE	Ponto Final
CURVE TYPE	Lineal
	Suprimida
UNITS	µg/dL
NO. OF DECIMAL PLACES	1
PRIMARY WAVELENGTH	570 nm
SECONDARY WAVELENGTH	-
READ TIME/INTERVAL	10
SAMPLE BLANK	-
FACTOR	**
CALIBRATION INTERVAL	*
NORMALIZATION INTERVAL	-
NO. OF CALIBRATORS	2
NO. OF REPLICATES	2
LOW BLANK A LIMIT	-0.100
HIGH BLANK A LIMIT	0.800
LOW A LIMIT	-0.100
HIGH A LIMIT	2.400
LOW NORMAL	#
HIGH NORMAL	#
LINEARITY LIMIT	500.0 µg/dL
CURVES S.D. LIMIT	-

TEST NAME	FE BI
TEST	FEBI
TEST BAR CODE	#
SAMPLE VOLUME	30 µL
SAMPLE DILUENT	-
PREDILUTION RATIO	1
RERUN DILUTION RATIO	1
PREDILUTION	-
REAGENT DILUENT	-
REAGENT 1 VOLUME	220 µL
DILUENT VOLUME	-
BAR CODE	FEB1
LAG TIME	290 seg.
REAGENT 2 VOLUME	15 µL
DILUENT VOLUME	-
BAR CODE	FEB2
LAG TIME	300

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TEST NAME	FERRO CAB
TEST	FECB
TEST BAR CODE	#
TEST TYPE	Ponto Final
CURVE TYPE	Lineal
	Suprimida
UNITS	µg/dL
NO. OF DECIMAL PLACES	1
PRIMARY WAVELENGTH	600 nm
SECONDARY WAVELENGTH	-
READ TIME/INTERVAL	20
SAMPLE BLANK	-
FACTOR	**
CALIBRATION INTERVAL	*
NORMALIZATION INTERVAL	-
NO. OF CALIBRATORS	2
NO. OF REPLICATES	2
LOW BLANK A LIMIT	-0.100
HIGH BLANK A LIMIT	0.500
LOW A LIMIT	-0.100
HIGH A LIMIT	2.400
LOW NORMAL	#
HIGH NORMAL	#
LINEARITY LIMIT	500.0 µg/dL
CURVES S.D. LIMIT	20

TEST NAME	FERRO CAB
TEST	FECB
TEST BAR CODE	#
SAMPLE VOLUME	15 µL
SAMPLE DILUENT	-
PREDILUTION RATIO	-
RERUN DILUTION RATIO	1
PREDILUTION	-
REAGENT DILUENT	-
REAGENT 1 VOLUME	300 µL
DILUENT VOLUME	-
BAR CODE	FECB
LAG TIME	300 seg.
REAGENT 2 VOLUME	-
DILUENT VOLUME	-
BAR CODE	-
LAG TIME	-

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TEST NAME	FÓSFORO
TEST	FOSF
TEST BAR CODE	#
TEST TYPE	Ponto Final
CURVE TYPE	Lineal Suprimida
UNITS	mg/dL
NO. OF DECIMAL PLACES	1
PRIMARY WAVELENGTH	340 nm
SECONDARY WAVELENGTH	-
READ TIME/INTERVAL	20
SAMPLE BLANK	-
FACTOR	**
CALIBRATION INTERVAL	*
NORMALIZATION INTERVAL	-
NO. OF CALIBRATORS	2
NO. OF REPLICATES	2
LOW BLANK A LIMIT	-0.100
HIGH BLANK A LIMIT	0.600
LOW A LIMIT	-0.100
HIGH A LIMIT	2.400
LOW NORMAL	#
HIGH NORMAL	#
LINEARITY LIMIT	20.0 µg/dL
CURVES S.D. LIMIT	0.25

TEST NAME	FÓSFORO
TEST	FOSF
TEST BAR CODE	#
SAMPLE VOLUME	3 µL
SAMPLE DILUENT	-
PREDILUTION RATIO	2
RERUN DILUTION RATIO	1
PREDILUTION	-
REAGENT DILUENT	-
REAGENT 1 VOLUME	300 µL
DILUENT VOLUME	-
BAR CODE	FOS1
LAG TIME	120 seg.
REAGENT 2 VOLUME	-
DILUENT VOLUME	-
BAR CODE	-
LAG TIME	-

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TEST NAME	FRUTOSAMINA
TEST	FRUT
TEST BAR CODE	#
TEST TYPE	Dois Pontos
CURVE TYPE	Blanked Linear
UNITS	mmol/L
NO. OF DECIMAL PLACES	1
PRIMARY WAVELENGTH	546 nm
SECONDARY WAVELENGTH	-
READ TIME/INTERVAL	180
SAMPLE BLANK	Não
FACTOR	4300**
CALIBRATION INTERVAL	*
NORMALIZATION INTERVAL	-
NO. OF CALIBRATORS	2
NO. OF REPLICATES	2
LOW BLANK A LIMIT	-0.100
HIGH BLANK A LIMIT	2.500
LOW A LIMIT	-0.100
HIGH A LIMIT	2.400
LOW NORMAL	#
HIGH NORMAL	#
LINEARITY LIMIT	8. mmol/L
CURVES S.D. LIMIT	-

TEST NAME	FRUTOSAMINA
TEST	FRUT
TEST BAR CODE	#
SAMPLE VOLUME	12 µL
SAMPLE DILUENT	-
PREDILUTION RATIO	1
RERUN DILUTION RATIO	1
PREDILUTION	-
REAGENT DILUENT	-
REAGENT 1 VOLUME	240 µL
DILUENT VOLUME	-
BAR CODE	FRT1
LAG TIME	400 seg.
REAGENT 2 VOLUME	-
DILUENT VOLUME	-
BAR CODE	-
LAG TIME	-

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TEST NAME	GLICOSE
TEST	GLI
TEST BAR CODE	#
TEST TYPE	Ponto Final
CURVE TYPE	Lineal
	Suprimida
UNITS	mg/dL
NO. OF DECIMAL PLACES	1
PRIMARY WAVELENGTH	510 nm
SECONDARY WAVELENGTH	-
READ TIME/INTERVAL	20
SAMPLE BLANK	-
FACTOR	#
CALIBRATION INTERVAL	#
NORMALIZATION INTERVAL	-
NO. OF CALIBRATORS	2
NO. OF REPLICATES	2
LOW BLANK A LIMIT	-0.100
HIGH BLANK A LIMIT	0.500
LOW A LIMIT	-0.100
HIGH A LIMIT	2.400
LOW NORMAL	#
HIGH NORMAL	#
LINEARITY LIMIT	mg/dL
CURVES S.D. LIMIT	10

TEST NAME	GLICOSE
TEST	GLI
TEST BAR CODE	#
SAMPLE VOLUME	3 µL
SAMPLE DILUENT	-
PREDILUTION RATIO	1
RERUN DILUTION RATIO	1
PREDILUTION	-
REAGENT DILUENT	-
REAGENT 1 VOLUME	300 µL
DILUENT VOLUME	-
BAR CODE	GL1B
LAG TIME	300 seg.
REAGENT 2 VOLUME	-
DILUENT VOLUME	-
BAR CODE	-
LAG TIME	-

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TEST NAME	LACTATO
TEST	LAC
TEST BAR CODE	#
TEST TYPE	Ponto Final
CURVE TYPE	Lineal
	Suprimida
UNITS	mg/dL
NO. OF DECIMAL PLACES	1
PRIMARY WAVELENGTH	546 nm
SECONDARY WAVELENGTH	-
READ TIME/INTERVAL	5
SAMPLE BLANK	-
FACTOR	**
CALIBRATION INTERVAL	*
NORMALIZATION INTERVAL	-
NO. OF CALIBRATORS	2
NO. OF REPLICATES	2
LOW BLANK A LIMIT	-0.100
HIGH BLANK A LIMIT	0.300
LOW A LIMIT	-0.500
HIGH A LIMIT	1.100
LOW NORMAL	#
HIGH NORMAL	#
LINEARITY LIMIT	120.0 mg/dL
CURVES S.D. LIMIT	-

TEST NAME	LACTATO
TEST	LAC
TEST BAR CODE	#
SAMPLE VOLUME	3 µL
SAMPLE DILUENT	-
PREDILUTION RATIO	1
RERUN DILUTION RATIO	1
PREDILUTION	-
REAGENT DILUENT	-
REAGENT 1 VOLUME	300 µL
DILUENT VOLUME	-
BAR CODE	LAC1
LAG TIME	300 seg.
REAGENT 2 VOLUME	-
DILUENT VOLUME	-
BAR CODE	-
LAG TIME	-

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TEST NAME	MAGNÉSIO
TEST	MG
TEST BAR CODE	#
TEST TYPE	Ponto Final
CURVE TYPE	Lineal Suprimida
UNITS	mg/dL
NO. OF DECIMAL PLACES	1
PRIMARY WAVELENGTH	510 nm
SECONDARY WAVELENGTH	-
READ TIME/INTERVAL	20
SAMPLE BLANK	-
FACTOR	**
CALIBRATION INTERVAL	*
NORMALIZATION INTERVAL	-
NO. OF CALIBRATORS	2
NO. OF REPLICATES	2
LOW BLANK A LIMIT	-0.100
HIGH BLANK A LIMIT	0.500
LOW A LIMIT	-0.100
HIGH A LIMIT	2.400
LOW NORMAL	#
HIGH NORMAL	#
LINEARITY LIMIT	3.5 mg/dL
CURVES S.D. LIMIT	0.500

TEST NAME	MAGNÉSIO
TEST	MG
TEST BAR CODE	#
SAMPLE VOLUME	3 µL
SAMPLE DILUENT	-
PREDILUTION RATIO	-
RERUN DILUTION RATIO	1
PREDILUTION	-
REAGENT DILUENT	-
REAGENT 1 VOLUME	300 µL
DILUENT VOLUME	-
BAR CODE	MG1
LAG TIME	120 seg.
REAGENT 2 VOLUME	-
DILUENT VOLUME	-
BAR CODE	-
LAG TIME	-

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TEST NAME	PROTEÍNA TOTAL
TEST	PROT
TEST BAR CODE	#
TEST TYPE	Ponto Final
CURVE TYPE	Lineal
	Suprimida
UNITS	g/dL
NO. OF DECIMAL PLACES	1
PRIMARY WAVELENGTH	540 nm
SECONDARY WAVELENGTH	-
READ TIME/INTERVAL	20
SAMPLE BLANK	-
FACTOR	**
CALIBRATION INTERVAL	*
NORMALIZATION INTERVAL	-
NO. OF CALIBRATORS	2
NO. OF REPLICATES	2
LOW BLANK A LIMIT	-0.100
HIGH BLANK A LIMIT	0.500
LOW A LIMIT	-0.100
HIGH A LIMIT	2.400
LOW NORMAL	#
HIGH NORMAL	#
LINEARITY LIMIT	12 g/dL
CURVES S.D. LIMIT	0.300

TEST NAME	PROTEÍNA TOTAL
TEST	PROT
TEST BAR CODE	#
SAMPLE VOLUME	6 µL
SAMPLE DILUENT	-
PREDILUTION RATIO	1
RERUN DILUTION RATIO	2
PREDILUTION	-
REAGENT DILUENT	-
REAGENT 1 VOLUME	300 µL
DILUENT VOLUME	-
BAR CODE	PTT
LAG TIME	300 seg.
REAGENT 2 VOLUME	-
DILUENT VOLUME	-
BAR CODE	-
LAG TIME	-

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TEST NAME	TRIGLICÉRIDES
TEST	TRI
TEST BAR CODE	#
TEST TYPE	Ponto Final
CURVE TYPE	Lineal Suprimida
UNITS	mg/dL
NO. OF DECIMAL PLACES	1
PRIMARY WAVELENGTH	510 nm
SECONDARY WAVELENGTH	-
READ TIME/INTERVAL	20
SAMPLE BLANK	-
FACTOR	**
CALIBRATION INTERVAL	*
NORMALIZATION INTERVAL	-
NO. OF CALIBRATORS	2
NO. OF REPLICATES	2
LOW BLANK A LIMIT	-0.100
HIGH BLANK A LIMIT	0.500
LOW A LIMIT	-0.100
HIGH A LIMIT	2.400
LOW NORMAL	#
HIGH NORMAL	#
LINEARITY LIMIT	1100.0 mg/dL
CURVES S.D. LIMIT	10

TEST NAME	TRIGLICÉRIDES
TEST	TRI
TEST BAR CODE	#
SAMPLE VOLUME	3 µL
SAMPLE DILUENT	-
PREDILUTION RATIO	2
RERUN DILUTION RATIO	1
PREDILUTION	-
REAGENT DILUENT	-
REAGENT 1 VOLUME	300 µL
DILUENT VOLUME	-
BAR CODE	TR1B
LAG TIME	300 seg.
REAGENT 2 VOLUME	-
DILUENT VOLUME	-
BAR CODE	-
LAG TIME	-

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TEST NAME	URÉIA
TEST	URÉIA
TEST BAR CODE	#
TEST TYPE	Cinético
CURVE TYPE	Lineal
	Suprimida
UNITS	mg/dL
NO. OF DECIMAL PLACES	1
PRIMARY WAVELENGTH	340 nm
SECONDARY WAVELENGTH	-
READ TIME/INTERVAL	60 seg
SAMPLE BLANK	-
FACTOR	#
CALIBRATION INTERVAL	-
NORMALIZATION INTERVAL	-
NO. OF CALIBRATORS	2
NO. OF REPLICATES	2
LOW BLANK A LIMIT	-0.100
HIGH BLANK A LIMIT	2.000
LOW A LIMIT	-0.100
HIGH A LIMIT	2.400
LOW NORMAL	#
HIGH NORMAL	#
LINEARITY LIMIT	300.0 mg/dL
CURVES S.D. LIMIT	4.0

TEST NAME	URÉIA
TEST	URÉIA
TEST BAR CODE	#
SAMPLE VOLUME	3 µL
SAMPLE DILUENT	-
PREDILUTION RATIO	1
RERUN DILUTION RATIO	1
PREDILUTION	-
REAGENT DILUENT	-
REAGENT 1 VOLUME	300 µL
DILUENT VOLUME	-
BAR CODE	UR1A
LAG TIME	60 seg.
REAGENT 2 VOLUME	-
DILUENT VOLUME	-
BAR CODE	-
LAG TIME	-

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